UWAGA

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ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist.

When used for the first time, the support should be put on by the experienced health-care personnel.

Before the first use, please, read the user's manual carefully.

If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

Indications:

Post-traumatic conditions of ankle joint (sprain, dislocation) during functional treatment.

Instability of ankle joint.

Stabilization after surgeries.

Ankle joint protection from recurrent injuries while re-undertaking occupational or sport activities after experiencing injury.

Contraindications:

Conditions which require immobilization of ankle joint.

Difficult to heal wounds, sores and other skin changes.

Dysesthesia.

Product description:

Stabilizing brace provides optimal comfort of use – plantar band of regulated length allows to adjust the circumference and ensures unhampered dorsiflexion/plantarflexion movement. Part of external insert of the brace is filled with gel which can be cooled and function as a cold compress to prevent oedema, inflammation and to alleviate pain

Use:

Never put the brace on a bare foot (the best option is to put the brace on a soft, cotton sock) which allows to avoid skin irritation and abrasion.

Unfasten the stabilizing bands.

Put on the brace so that the junctive band would be under the foot and two parts of brace embrace the lateral ankle and medial ankle (using a Velcro band adjust the width of junctive band).

Make sure that the brace is placed parallel to longitudinal axis of lower limb.

Fasten the stabilizing bands around the ankle joint so that the brace would adhere to the body but would not restrict circulation.

If you want to use the gel insert as a cold compress, put it into the refrigerator before using. Do not freeze the insert – it may lead to frost-caused injuries.

Maintenance:

Clean the brace with moist cloth using mild detergents.

ATTENTION

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

Universal size