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



English version on next page

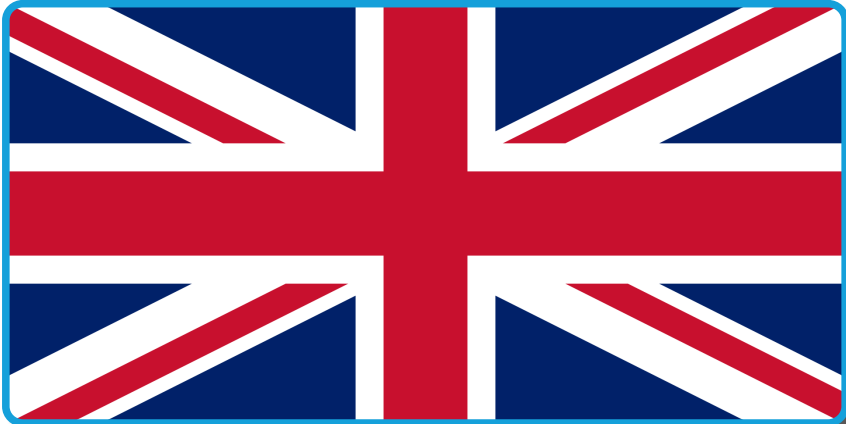WARNING!


The user should be aware that the surface of the remote control may get heat up when it is exposed to sunlight for an extended period of time.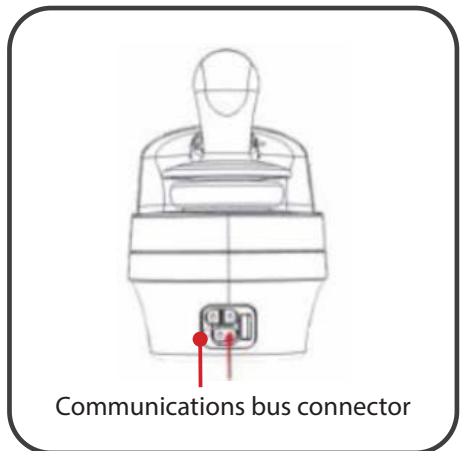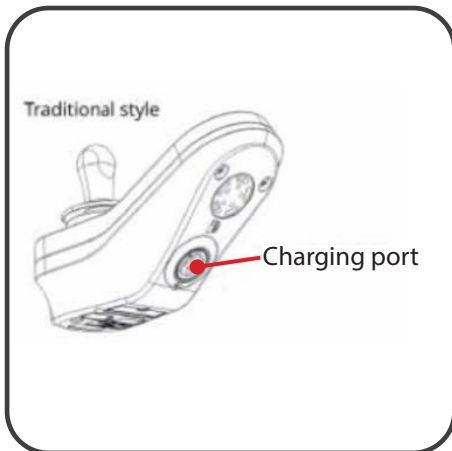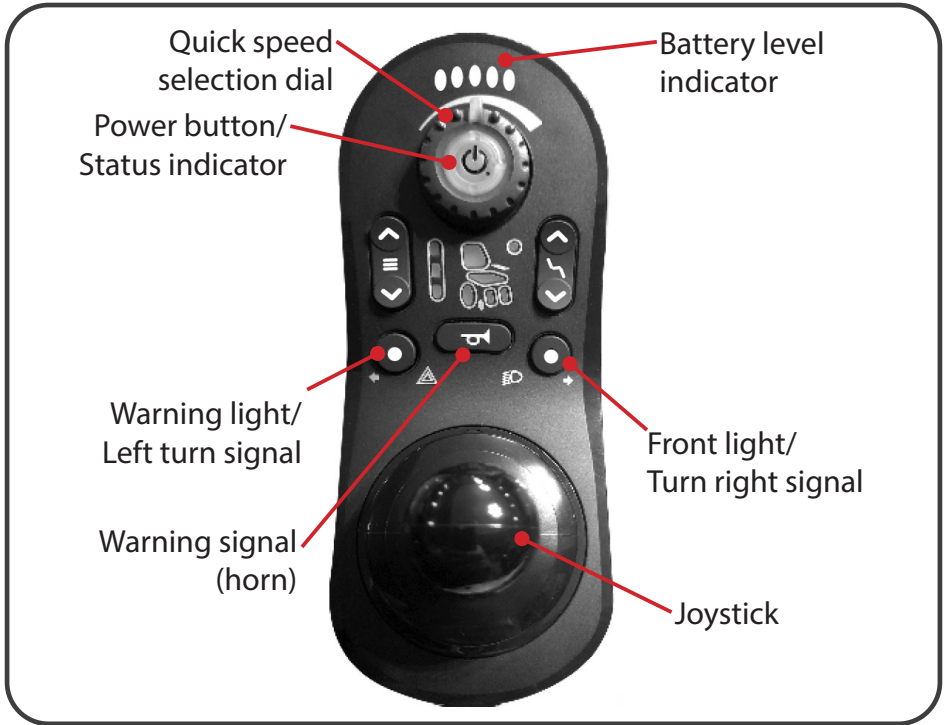


Control panel - operation

Adjustment of wheelchair function operation

Quick speed selection dial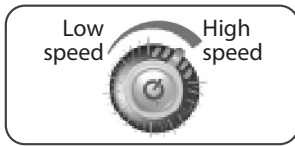


To set the lowest speed, turn the dial as far to the left as possible.
To set the highest speed, turn the dial as far to the right as possible.

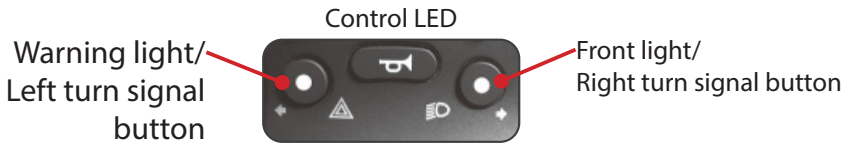Indicator lamps and displayed information

To turn on the emergency light, press the button once. To turn off the emergency light, press the left or right button once.

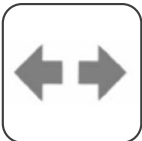Turn signal lights:

To turn on left (right) signal lights, please press left (or right) lighting button for a while (3 seconds) and release. Left (or right) side LED will light up.

To turn off signal lights, please press left or right lighting button once.

Front lights:

To turn on the headlight, press the button on the controller once. To turn off the headlight, please press once, the left or right front light or turn signal button.

DISPOSAL OF WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT

This symbol on the products or accompanying documents means that used electrical or electronic products should not be disposed of with the normal household waste. For proper disposal, refurbishment or recycling, please take these products to the collection points for this type of waste.

Alternatively, in some EU countries or other European countries, you may return your product to your local retailer when purchasing a similar new product. By disposing of this product correctly, you will help to conserve valuable natural resources and support the prevention of potential negative effects on the environment and human health, which could result from incorrect disposal. For further information, please contact your local municipality or collection point waste collection points. Penalties may be imposed in accordance with local regulations if this product is disposed of in an inappropriate manner. For parties within the European Union, if you wish to discard electrical or electronic equipment, please obtain the necessary information from your retailer or supplier.

Elimination in countries outside the European Union. This symbol is valid in the European Union. If you wish to decommission this appliance, please obtain the correct decommissioning information from your local authorities or from your retailer.

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Polska

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl

www.viteacare.com